



## Ring medical shower BRYZA type NPLE

Link to product:


<http://meden.com.pl/offer/hydrotherapy-medical-equipment/9-ring-medical-shower-bryza-type-nple.html>

Available online only

Condition: New

### Informacje:

Shower cabin Bryza is dedicated to conduct a therapeutic procedure using water massaging properties and temperature.

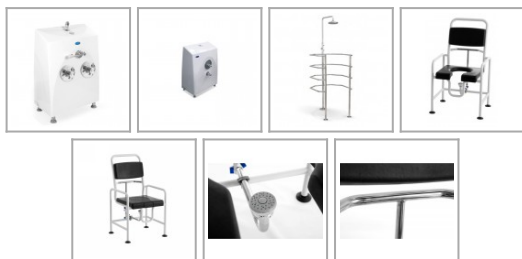

### Product description:

## Bryza Ring showers

**Bryza** is dedicated to conduct a therapeutic procedure using water massaging properties and temperature. Appropriately arranged water jets, spray water under high pressure, massaging the patient's body uniformly. Console **Bryza** allows the therapist to control the water pressure and temperature, whereby the treatment can influence the patient in different ways. Popular procedures include alternately showers during the treatment a rapid change of temperature is used.

### Product features:

- Free-standing console made from gelcoat reinforced with glass fiber
- Temperature sensor
- Independent smooth temperature control of cold and warm water (2 thermostates)
- Quick changes of temperature between cold and warm water (2 electrovalves)
- Electronic control panel with temperature indication
- Console has to be permanently connected to the electric system
- 3 standard programs, 4 user programs
- Compensation tank for reduction of small water pressure fluctuations in the water system

### Additional options:

**PUMP** - The pump stabilizing pressure for use in case of low pressure in the water supply network.

### Treatment possibilities:

- Cold and cool showers - temp. 10-20°C. From a few seconds to a minute. Toughen the body, stimulate metabolism and raise the blood pressure. Often used after hot hydrotherapy procedures, like balneological bath or sauna.
- Warm showers - temp. 32-37°C; duration 3-5 min. Recommended in neurosis, sleeplessness and neuralgias.
- Hot showers - temp. 38-42°C; duration 5-10 min. Stimulate metabolism, increase sweating and vasodilation, lower the blood pressure, provide general relaxation and sedation.
- Cold / warm showers - temp. 32-37°C for approx. 1 minute, then 10-20°C for approx. 30 sec.; duration 5 min. Toughen the body, relax and stimulate metabolism

---

**Cage features:**

- Piping made from corrosion-resistant materials
  - Operated from a dedicated console NPL2 or NPLE
  - 2 independent zones: shower from above or side shower
- 

**Chair features:**

- Made of: anti- corrosion protected, powder coated steel and stainless steel
- Seat and back rest made of waterproof material
- Operated from special adjusted console to work with chair

**Technical data:**

<b>Cold and warm water pressure [bar]:</b>	4-6
<b>Warm water temperature [°C]:</b>	max. 60
<b>Power supply (applicable to NPLE [V/Hz]:</b>	230/50
<b>Power consumption [A]:</b>	5
<b>Dimensions NPL2 (H x W x L) [cm]:</b>	100 x 64 x 56
<b>Dimensions NPLE (H x W x L) [cm]:</b>	100 x 67 x 55
<b>Dimensions CAGE (H x W x L) [cm]:</b>	216 x 89 x 89
<b>Dimensions CHAIR (H x W x L) [cm]:</b>	96 x 60 x 59