

Bryza is dedicated to conduct a therapeutic procedure using water massaging properties and temperature. Appropriately arranged water jets, spray water under high pressure, massaging the patient's body uniformly. Console **Bryza** allows the therapist to control the water pressure and temperature, whereby the treatment can influence the patient in different ways. Popular procedures include alternately showers during the treatment a rapid change of temperature is used.

Treatment possibilities:

- Cold and cool showers - temp. 10-20°C. From a few seconds to a minute. Toughen the body, stimulate metabolism and raise the blood pressure. Often used after hot hydrotherapy procedures, like balneological bath or sauna.
- Warm showers - temp. 32-37°C; duration 3-5 min. Recommended in neurosis, sleeplessness and neuralgias.
- Hot showers - temp. 38-42°C; duration 5-10 min. Stimulate metabolism, increase sweating and vasodilation, lower the blood pressure, provide general relaxation and sedation.
- Cold / warm showers - temp. 32-37°C for approx. 1 minute, then 10-20°C for approx. 30 sec.; duration 5 min. Toughen the body, relax and stimulate metabolism

Cage features:

- Piping made from corrosion-resistant materials
- Operated from a dedicated console NPL2 or NPLE
- 2 independent zones: shower from above or side shower


Chair features:

- Made of: anti-corrosion protected, powder coated steel and stainless steel
- Seat and back rest made of waterproof material
- Operated from special adjusted console to work with chair

Technical data:

Cold and warm water pressure [bar]:	4-6
Warm water temperature [°C]:	max. 60
Power supply (applicable to NPLE [V/Hz]):	230/50
Power consumption [A]:	5
Dimensions NPL2 (H x W x L) [cm]:	100 x 64 x 56
Dimensions NPLE (H x W x L) [cm]:	100 x 67 x 55
Dimensions CAGE (H x W x L) [cm]:	216 x 89 x 89
Dimensions CHAIR (H x W x L) [cm]:	96 x 60 x 59

 Packing dimensions (cage+NPL2/NPLE): **120 x 100 x 165 cm**, Weight: **76 kg**

 Packing dimensions (console+chair): **120 x 80 x 117 cm**, Weight: **73 kg**

Bryza type NPL2 features:

- Free-standing console made from gelcoat reinforced with glass fiber
- Temperature control of one water source (1 thermostat)
- Lever for fast temperature changes
- Valve to regulate pressure and cut off the water
- Compensation tank for the reduction of small water pressure fluctuations in the water system
- Thermometer

Bryza type NPLE* features:

- Free-standing console made from gelcoat reinforced with glass fiber
- Temperature sensor
- Independent smooth temperature control of cold and warm water (2 thermostats)
- Quick changes of temperature between cold and warm water (2 electrovalves)
- Electronic control panel with temperature indication
- Console has to be permanently connected to the electric system
- 3 standard programs, 4 user programs
- Compensation tank for reduction of small water pressure fluctuations in the water system

*There is possible NPLE with PUMP stabilizing pressure

